


MYCOPLASMA SYSTEM Vet

Count, Identification and Susceptibility Testing of urogenital mycoplasmas and *Candida* spp./*Trichomonas vaginalis* directly from Veterinary Specimens

Liofilchem® and the Liofilchem company logo are registered trademarks of LIOFILCHEM s.r.l.

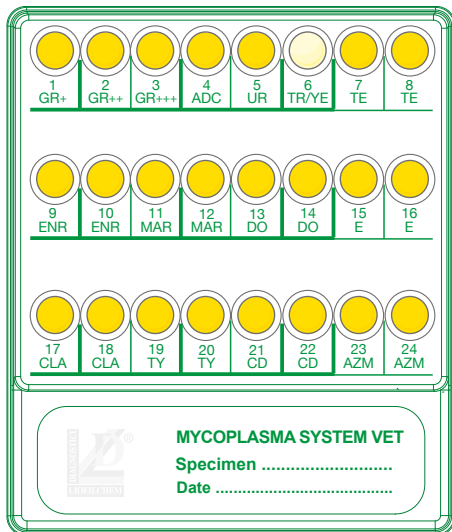

★ Veterinary Specimens

★ Urogenital mycoplasmas


Count/identification

Ureaplasma urealyticum, *Mycoplasma* spp.

Susceptibility Testing

(double concentration)

Tetracycline, *Enrofloxacin*, *Marbofloxacin*,
Doxycycline, *Erythromycin*, *Clarithromycin*,
Tylosin, *Clindamycin*, *Azithromycin*

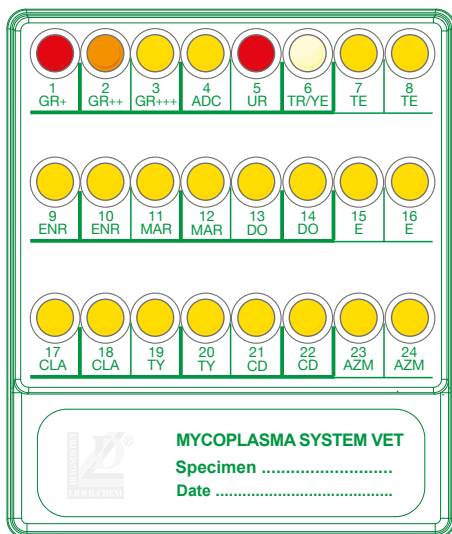


★ Direct Microbial Identification

(microscopic observation)

Candida spp.,
Trichomonas vaginalis

★ Benefits

- ✓ Complete diagnosis
- ✓ Save time and labor with direct testing
- ✓ Results in 24-48 hours
- ✓ Clear and well defined color changes
- ✓ Easy interpretation
- ✓ Automatic reading available

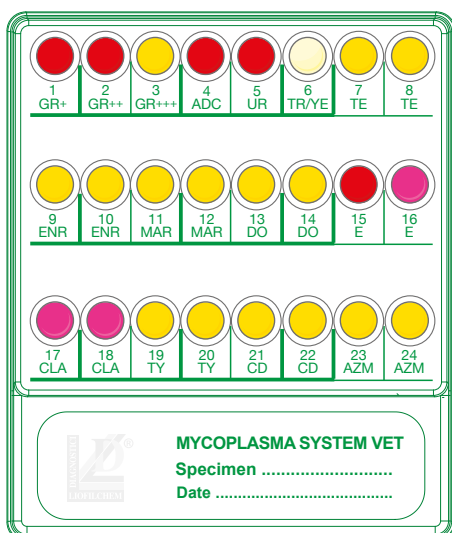


www.liofilchem.net

MYCOPLASMA SYSTEM Vet


RESULTS

Urogenital mycoplasmas


Count

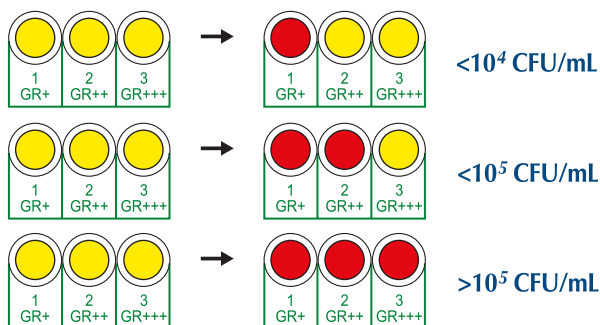


Identification

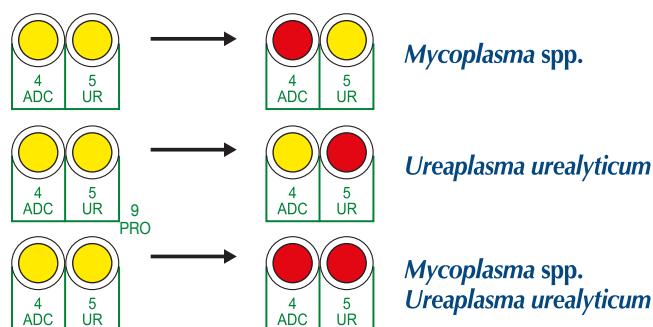

Susceptibility Testing (double concentration)

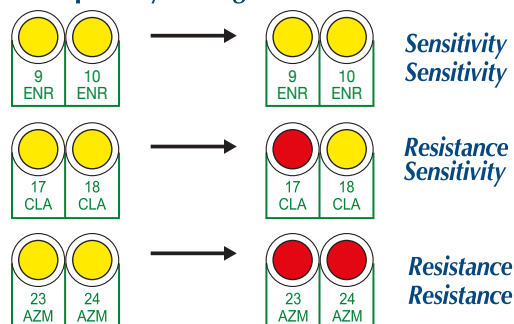

Microscopic observation (400x)

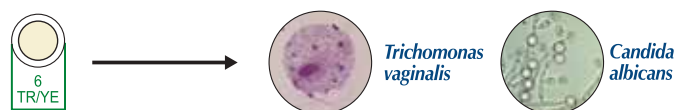

DESCRIPTION	PACKAGING	REF.
MYCOPLASMA SYSTEM Vet	20 tests	72593
VASELINE OIL	2x50 mL	80278

TEST PROCEDURE



1

- Dip the swab into the vial of physiological solution and wait 5 minutes. Carefully squeeze the swab against the vial wall.



2

- Dispense 0.2 mL of clinical sample suspension into each well of the system.



3

- Cover all the wells with 1 drop of Vaseline Oil, except 6-TR/YE well. Cover the system with the lid provided and incubate in thermostat at 36±1°C for 18-24 hours.



4

- Confirm *Trichomonas vaginalis* and *Candida* spp. by taking a drop from well 6-TR/YE and watch at the microscope (400x).



5

- Watch for the color change of the wells and interpret the results.



6

- Note down the test results on the form provided in the kit, or read and interpret with automatic readers.