

# ComASP®

## Compact Antimicrobial Susceptibility Panel

for determining MIC through broth microdilution according to ISO 20776-1:2019 standard

Description	Packaging	Ref.
<b>ComASP® Colistin 0.25-16</b> Compact Antimicrobial Susceptibility Panel containing colistin (polymyxin E) in 7 two-fold dilutions (0.25-16 µg/mL), that allows for testing up to four samples per panel.	4 panels (16 tests)	75001
<b>ComASP® Cefiderocol 0.008-128</b> Compact Antimicrobial Susceptibility Panel containing cefiderocol in 15 two-fold dilutions (0.008-128 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75009
<b>ComASP® Oritavancin 0.001-16</b> Compact Antimicrobial Susceptibility Panel containing oritavancin in 15 two-fold dilutions (0.001-16 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75010
<b>ComASP® Benzylpenicillin 0.002-32</b> Compact Antimicrobial Susceptibility Panel containing Benzylpenicillin in 15 two-fold dilutions (0.002-32 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75011
<b>ComASP® Vancomycin 0.008-128</b> Compact Antimicrobial Susceptibility Panel containing vancomycin in 15 two-fold dilutions (0.008-128 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75007
<b>ComASP® Piperacillin-tazobactam 0.008/4-128/4</b> Compact Antimicrobial Susceptibility Panel containing piperacillin-tazobactam in 15 two-fold dilutions (0.008/4-128/4 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75002
<b>ComASP® Colistin / Piperacillin-tazobactam</b> Compact Antimicrobial Susceptibility Panel containing the antibiotics in 15 two-fold dilutions, that allows for testing one sample per panel to colistin (0.008-128 µg/mL) and piperacillin-tazobactam (0.008/4-128/4 µg/mL).	4 panels (4 tests)	75003
<b>ComASP® Ceftolozane-tazobactam 0.008/4-128/4</b> Compact Antimicrobial Susceptibility Panel containing ceftolozane-tazobactam in 15 two-fold dilutions (0.008/4-128/4 µg/mL), that allows for testing up to two samples per panel.	4 panels (8 tests)	75006
<b>ComASP® Ceftolozane-tazobactam / Ceftazidime-avibactam</b> Compact Antimicrobial Susceptibility Panel containing the antibiotics in 15 two-fold dilutions, that allows for testing one sample per panel to ceftolozane-tazobactam (0.008/4-128/4 µg/mL) and ceftazidime-avibactam (0.008/4-128/4 µg/mL).	4 panels (4 tests)	75004
<b>ComASP® Vancomycin / Teicoplanin</b> Compact Antimicrobial Susceptibility Panel containing the antibiotics in 15 two-fold dilutions, that allows for testing one sample per panel to vancomycin (0.008-128 µg/mL) and teicoplanin (0.008-128 µg/mL).	4 panels (4 tests)	75005
<b>ComASP® Antifungal</b> Compact Antimicrobial Susceptibility Panel containing antifungal agents in various concentration ranges, that allows for testing one sample per panel.	4 panels (4 tests)	75101



**Liofilchem srl**

Zona Industriale, 64026 Roseto degli Abruzzi (TE), Italy

Tel +39 0858930745

www.liofilchem.com

Headquarters, Manufacturing Site, International distribution


**Liofilchem, Inc.**

465 Waverley Oaks Rd. Suite 317

Waltham, MA 02452, United States

Tel 781-902-0312

US Distribution Center

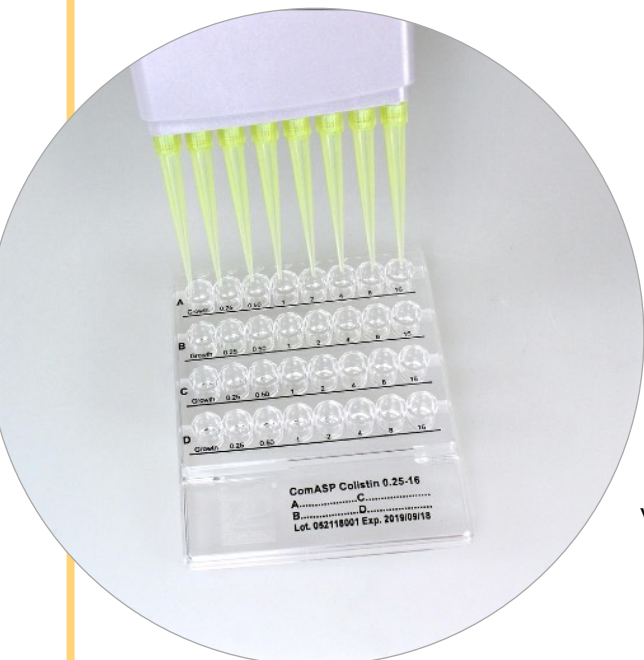

Incubation at  $36\pm 2^{\circ}\text{C}$  for 16-20 hours in ambient air.

At the end of the incubation period watch for the growth in the wells and determine the MIC, i.e. the lowest concentration of antibiotic that inhibits visible growth.



Growth appears as turbidity or as a button at the bottom of the well (compare with the amount of growth in the growth-control well).



The result can be read visually or automatically. Reading by the naked eye can be improved by use of bright indirect lighting against a dark background.

Liofilchem®, the Liofilchem company logo and ComASP® are registered trademarks of LIOFILCHEM s.r.l.



### Liofilchem srl

Zona Industriale, 64026 Roseto degli Abruzzi (TE), Italy  
Tel +39 0858930745  
www.liofilchem.com  
Headquarters, Manufacturing Site, International distribution

### Liofilchem, Inc.

465 Waverley Oaks Rd. Suite 317  
Waltham, MA 02452, USA  
Tel 781-902-0312  
US Distribution Center

ref. 6550221  
Rev. 9  
15 October 2022  
© Liofilchem 2022  
printed in Italy